

Handy length of 120 cm; fits all CPAP hoses (with inside diameter 15 mm and connector 22 mm).

Application

1. Carefully uncoil and straighten the hose brush. To avoid placing stress on the wire from coiling and uncoiling, we suggest finding a place to hang the brush.
2. Prior to use, wash the brush using a mild cleaning product (e.g. SomnoSept) and warm water. After that, rinse with cold water.
3. Likewise, use a mild cleaning product and lukewarm water for cleaning the CPAP hose (cold or lukewarm water is recommended, as hot water can dry out plastics and shorten their life).
4. Hold the hose as straight as possible to prevent unnecessary wear or damage. Gently push the CPAP brush further into the hose using short thrusting movements.
5. To clean the other end, insert the hose brush into the opposite end of the tubing.
6. After cleaning, rinse the hose thoroughly with cold water.
7. To dry the inside of the hose, connect it to the CPAP machine and let it run for about 15 minutes until the moisture has evaporated.
8. Carefully dry the brush bristles by dabbing them with a towel or cloth, then hang the brush up in a room with adequate air circulation.

Notes / Warnings

The brush is shorter than a CPAP tube. This ensures the brush is easy to handle. With this in mind, the CPAP hose should be cleaned from both sides using the brush. Store out of reach of small children. If the brush head is damaged, please discontinue use.